

Representative Reactivities Table



Optitrol NAT Screening

Positive Control

Version: 2

Date: June 30, 2021

Control	Manufacturer	Assay Name	Unit	Mean	SD	Samples	Labs	Representative Reactivity
Optitrol NAT HCV Screening	Roche	cobas® MPX Multiplex HIV, HCV & HBV nucleic acid test for use on cobas® 6800/8800 Systems	IU/mL	49.7	10.4	14	1	positive
	Grifols	Procleix® Ultrio Elite Assay Procleix Panther System	S/CO	23.6	1.0	30	3	positive
Optitrol NAT HBV Screening	Roche	cobas® MPX Multiplex HIV, HCV & HBV nucleic acid test for use on cobas® 6800/8800 Systems	IU/mL	48.30	15.0	14	1	positive
	Grifols	Procleix® Ultrio Elite Assay Procleix Panther System	S/CO	23.8	1.2	30	3	positive
Optitrol NAT HIV Screening	Roche	cobas® MPX Multiplex HIV, HCV & HBV nucleic acid test for use on cobas® 6800/8800 Systems	IU/mL	267.2	80.9	14	1	positive
	Grifols	Procleix® Ultrio Elite Assay Procleix Panther System	S/CO	12.1	0.6	30	3	positive

The stated mean values were obtained by testing one product batch. They show possible representative values but no target values. Test results may vary among used instruments, assays, reagent lots, and different laboratory protocols.