

## Representative Reactivities Table



### Optitrol NAT Viral Load

Positive Control

Version: 1

Date: December 18, 2020

Control	Manufacturer	Assay Name	Unit	Mean	SD	Samples	Labs	Representative Reactivity
Optitrol NAT HCV Viral Load	Roche	cobas® MPX Multiplex HIV, HCV & HBV nucleic acid test for use on cobas® 6800/8800 Systems	IU/ml	238.98	23.87	10	1	positive
Optitrol NAT HBV Viral Load	Roche	cobas® MPX Multiplex HIV, HCV & HBV nucleic acid test for use on cobas® 6800/8800 Systems	IU/ml	281.30	44.66	10	1	positive
Optitrol NAT HIV Viral Load	Roche	cobas® MPX Multiplex HIV, HCV & HBV nucleic acid test for use on cobas® 6800/8800 Systems	IU/ml	227.13	85.90	10	1	positive

The stated mean values were obtained by testing one product batch. They show possible representative values but no target values. Test results may vary among used instruments, assays, reagent lots, and different laboratory protocols.

