

## Representative Reactivities Table



### Optitrol Green

#### Positive Control

Analyte	Manufacturer	Assay Name	Method	Unit	Mean	SD	Samples	Labs	Representative Reactivity
Anti-HCV IgG	Bio-Rad	Monolisa HCV ULTRA V2	EIA	S/CO	3.83	0.33	39	4	positive
Anti-HIV 1 IgG	Bio-Rad	Genscreen ULTRA HIV Ag-Ab	EIA	S/CO	2.91	0.87	39	4	positive
HBsAg	Bio-Rad	Monolisa HBsAg Ultra	EIA (manual)	S/CO	6.61	1.37	6	1	positive
HBsAg	Bio-Rad	Monolisa HBsAg Ultra	EIA (Evolis)	S/CO	31.3	9.27	31	3	positive
<i>Anti-Treponema pallidum</i> IgG	Bio-Rad	Syphilis Total Ab	EIA	S/CO	4.15	0.84	37	4	positive
Anti-HTLV I IgG	DiaSorin	Murex HTLV I + II	EIA	S/CO	3.03	0.35	18	2	positive

The stated mean values were obtained by testing one product batch. They show possible representative values but no target values. results may vary among used instruments, assays, reagent lots, and different laboratory protocols.

Test