

Representative Reactivities Table



Optitrol HIV 2

Positive Control

Version: 2

Date: March 16, 2021

Analyte	Manufacturer	Assay Name	Method	Unit	Mean	SD	Samples	Labs	Representative Reactivity
Anti-HIV 2 IgG	Abbott	Architect HIV Ag/Ab Combo	CMIA	S/CO	6.17	0.90	10	2	positive
Anti-HIV 2 IgG	Abbott	Alinity i HIV Ag/Ab Combo	CMIA	S/CO	5.28	0.26	10	2	positive
Anti-HIV 2 IgG	DiaSorin	LIAISON XL Murex HIV Ab/Ag	CLIA	S/CO	5.13	0.18	10	2	positive
Anti-HIV 2 IgG	Siemens	ADVIA Centaur HIV Ag/Ab Combo	ChLIA	Index	2.43	0.04	10	2	positive
Anti-HIV 2 IgG	Siemens	Atellica HIV Ag/Ab	ChLIA	Index	2.15	0.03	10	1	positive
Anti-HIV 2 IgG	Roche	Elecsys (cobas) HIV combi	ECLIA	COI	109	2.2	10	2	positive

The stated mean values were obtained by testing one product batch. They show possible representative values but no target values. Test results may vary among used instruments, assays, reagent lots, and different laboratory protocols.