

Representative Reactivities Table



Optitrol ToRCH G

Positive Control

Version: 1

Date: Jan 20, 2022

Analyte	Manufacturer	Assay Name	Method	Unit	Mean	SD	Samples	Labs	Representative Reactivity
Rubella IgG	Abbott	Architect Rubella IgG	CMIA	IU/mL	24.72	0.77	10	2	positive
	DiaSorin	Liaison Rubella IgG	CLIA	IU/ml	26.09	0.90	10	2	positive
	Vidas	Vidas RUB IgG II	ELFA	IE/mL	34.20	1.40	10	1	positive
Toxo IgG	Abbott	Architect Toxo IgG	CMIA	IU/mL	8.92	0.36	10	2	positive
	Abbott	Alinity i Toxo IgG	CMIA	IU/mL	8.85	0.56	10	1	positive
	DiaSorin	Liaison Toxo IgG II	CLIA	IU/mL	16.28	0.70	10	2	positive
	Vidas	Vidas Toxo IgG II	ELFA	IU/mL	14.90	1.10	10	1	positive
CMV IgG	Abbott	Architect CMV IgG	CMIA	AU/mL	45.14	1.75	10	2	positive
	Abbott	Alinity i CMV IgG	CMIA	AU/mL	50.06	2.61	10	1	positive
	DiaSorin	Liaison CMV IgG II	CLIA	U/mL	48.21	1.75	10	2	positive
	Vidas	Vidas CMV IgG	ELFA	AU/mL	24.20	1.14	10	1	positive
EBV VCA IgG	Abbott	Architect EBV VCA IgG	CMIA	S/CO	15.98	0.68	10	2	positive
	Abbott	Alinity i EBV VCA IgG	CMIA	S/CO	13.51	0.35	10	1	positive
	DiaSorin	Liaison VCA IgG	CLIA	U/mL	27.29	3.39	10	2	cutoff
	Vidas	Vidas EBV VCA/EA IgG	ELFA	TW	0.35	0.03	10	1	positive
EBNA IgG	Abbott	Architect EBNA-1 IgG	CMIA	S/CO	6.52	0.13	10	2	positive
	Abbott	Alinity i EBV EBNA-1 IgG	CMIA	S/CO	6.51	0.21	10	1	positive
	DiaSorin	Liaison EBNA IgG	CLIA	U/mL	43.27	1.19	10	2	positive
	Vidas	Vidas EBV EBNA IgG	ELFA	TW	1.16	0.11	10	1	positive
HSV-1/2 IgG	DiaSorin	Liaison HSV-1/2 IgG	CLIA	Index	13.87	0.44	10	2	positive

The stated mean values were obtained by testing one product batch. They show possible representative values but no target values. Test results may vary among used instruments, assays, reagent lots, and different laboratory protocols.