

## Representative Reactivities Table



### Optitrol ToRCH M

Positive Control

Version: 2

Date: February 9, 2022

Analyte	Manufacturer	Assay Name	Method	Unit	Mean	SD	Samples	Labs	Representative Reactivity
Anti-Rubella IgM	Abbott	Architect Rubella IgM	CMIA	Index	3.93	046	10	2	positive
	Abbott	Alinity i Rubella IgM	CMIA	Index	3.78	0.08	10	1	positive
	DiaSorin	Liaison Rubella IgM	CLIA	AU/ml	92.3	13.2	10	2	positive
	bioMérieux	VIDAS RUB IgM	ELFA	Index	2.98	0.03	10	1	positive
Anti-Toxo IgM	Abbott	Architect Toxo IgM	CMIA	Index	2.82	0.16	10	2	positive
	Abbott	Alinity i Toxo IgM	CMIA	Index	2.90	0.08	10	1	positive
	DiaSorin	Liaison Toxo IgM	CLIA	AU/ml	20.6	2.4	10	2	positive
	bioMérieux	VIDAS TOXO IgM	ELFA	Index	1.91	0.03	10	1	positive
Anti-CMV IgM	Abbott	Architect CMV IgM	CMIA	Index	5.12	0.14	10	2	positive
	Abbott	Alinity i CMV IgM	CMIA	Index	5.50	0.10	10	1	positive
	DiaSorin	Liaison CMV IgM	CLIA	U/ml	57.8	3.1	10	2	positive
	bioMérieux	VIDAS CMV IgM	ELFA	Index	1.58	0.04	10	1	positive
Anti-EBV VCA IgM	Abbott	Architect EBV VCA IgM	CMIA	S/CO	3.58	0.09	10	2	positive
	Abbott	Alinity i EBV VCA IgM	CMIA	S/CO	3.14	0.11	10	1	positive
	DiaSorin	Liaison EBV/VCA IgM	CLIA	U/ml	96.5	3.3	10	2	positive
	bioMérieux	VIDAS EBV VCA IgM	ELFA	Index	0.62	0.01	10	1	positive
Anti-HSV 1/2 IgM	DiaSorin	Liaison HSV 1/2 IgM	CLIA	Index	2.00	0.09	10	2	positive

The stated mean values were obtained by testing one product batch. They show possible representative values but no target values. Test results may vary among used instruments, assays, reagent lots, and different laboratory protocols.