

CMV IgM Seroconversion Panel

REF Catalogue No: SCP-CMV-001, Old Catalogue No: RP-003

The DiaMex GmbH CMV IgM Seroconversion Panel consists of 15 members with each member containing 0.5 mL of human plasma. This panel illustrates the onset and decline of IgM and IgG Cytomegalovirus antibodies from one individual over a period of 99 days.

1. Intended Use

This Seroconversion Panel (SCP) is intended for standard testing by diagnostic manufacturers and researchers during assay development, evaluation, troubleshooting and post-marked surveillance of IgM/IgG antibody test systems and methods. Moreover, it serves as validation tool for diagnostic sensitivity, determination of analytical sensitivity, identification of cut-off values or to study the humoral immune response to this infection.


The plasma has been defibrinated and filtered. Sodium azide (0.095%) was added as preservative. All panel members are ready to use.

2. Storage and Stability

Store the SCP at -20°C to -80°C. Thaw samples at room temperature and mix gently by inversion before usage. Avoid foaming, contamination and repeated freeze and thaw cycles. After usage, return immediately to storage conditions.

3. Warnings and Precautions

Potentially infectious materials. Handle the product as if capable of transmitting infectious diseases. Do not pipette by mouth.

 **WARNING:** contains sodium azide.

H302: Harmful if swallowed.

H413: May cause long lasting harmful effects to aquatic life.







Prevention:

P264: Wash hands thoroughly after handling.

P270: Do not eat, drink or smoke when using this product

P273: Avoid the release to the environment

Declaration of used symbols

 Catalog Number	 Lot Number	 Warning	 Consult Instructions for Use	 Manufactured by	 Temperature Limitation
---	---	--	---	--	---

Reaction:

P330: Rinse mouth.

P301 + P312 IF SWALLOWED: Call a POISON CENTER or doctor/physician if you feel unwell.

Disposal:

P501: Dispose of waste in accordance to applicable local or national regulations. Waste must be disposed in a secured manner.

4. Donor Information

All panel members have been tested and found negative/non-reactive for anti-HIV, anti-HCV, HBsAg and Syphilis with FDA approved tests.

Donor profile

- Sex: Female
- Age: 20
- Ethnicity: Hispanic/Caucasian
- Residence: U.S. West Coast
- Children: yes

Suspected mode of transmission

- Mother-child interaction

Medical background

- No recent medication
- No recent illness or change in lifestyle
- No history of high risk behaviour
- No history of surgery
- No history of hepatitis or liver disease
- No history of blood transfusions
- No history of drug or alcohol abuse

5. Detection Methods

Each panel member is assayed for IgM and IgG antibodies with Abbott ARCHITECT®, Roche ELECSYS® and Diasorin LIAISON® test devices. Testing is performed with CE marked test.

6. Limitations and Restrictions

This panel is for Research Use Only and not intended for human or animal diagnostics, or for therapeutic purposes. Each laboratory has the responsibility to ascertain the suitability of the SCP for its particular application and to establish their own guidelines for interpretation of results. Data is provided for informational purposes only. The DiaMex GmbH does not claim that others can duplicate these test results exactly.

CMV IgM Seroconversion Panel, Catalogue No: SCP-CMV-001

Panel Member	Day	Abbott Architect CMV IgM Index	Abbott Architect CMV IgG AU/mL	Roche Elecsys CMV IgM COI	Roche Elecsys CMV IgG U/mL	Diasorin Liaison CMV IgM II U/mL	Diasorin Liaison CMV IgG II U/mL
		cut off 1.0 Index	cut off 6.0 AU/mL	cut off 1.0 COI	cut off 1.0 U/mL	cut off 22 U/mL	cut off 14 U/mL
1	1	1.66	8.4	0.39	<0.15	54.5	9.4
2	4	5.43	16.4	1.09	<0.15	93.5	22.1
3	8	6.68	38.8	3.22	2.03	>140	49.2
4	51	1.82	80.2	1.37	5.87	61.8	66.9
5	55	1.76	81.5	1.48	7.42	54.5	69.7
6	59	1.72	86.4	1.42	8.16	47.2	68.3
7	65	1.32	80.0	1.51	8.11	40.8	62.6
8	67	1.42	80.4	1.56	8.57	38.2	61.5
9	72	1.29	81.4	1.57	9.41	31.2	58.2
10	74	1.33	82.2	1.55	9.61	28.7	57.5
11	79	1.27	80.4	1.49	12.52	30.1	57.1
12	84	1.32	93.2	1.58	16.01	33.2	61.1
13	88	1.37	100.7	1.76	20.08	35.8	64.5
14	95	1.23	93.2	1.68	22.43	29.3	53.1
15	99	1.22	92.0	1.67	22.60	28.9	55.6

Assay information:

Abbott Architect CMV IgM assay; Lot: 52129LF00; Exp. Date: 13.04.2016; Test Date: 11.08.2015

Abbott Architect CMV IgG assay; Lot: 51027LF00; Exp. Date: 04.03.2016; Test Date: 11.08.2015

Roche Elecsys CMV IgM assay; Lot: 17091200

Roche Elecsys CMV IgG assay; Lot: 16971900

Diasorin Liaison CMV IgM II assay; Lot: 167019X/1

Diasorin Liaison CMV IgG II assay; Lot: 161021X/1

CMV IgM Seroconversion Panel, Catalogue No: SCP-CMV-001

